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EN ATTENTION!!!

Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. We advise against re-utilization.

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EN Applicable to the following commercial models:

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EN 1.1 Product description:

Implacil Mold Transfers are pin-shaped components, screwed on to prosthetic dental implants or components in order to transfer the position of the same in the patient's mouth to a working model, using a mold.

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EN 2. Indication: Mold Transfer is indicated for cases of oral rehabilitation where there is a need to reproduce the positioning of the implant or prosthetic Abutment in the patient's mouth for the model to be worked on at the dental surgery or laboratory. Each Transfer is designed for a specific type of implant or prosthetic Abutment.

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EN 3. Special conditions for product transport, storage and packing: Storage: Store away from direct sun and rain and in its original packaging. Transport: During transport protect the packaging from direct rain. Packing: Keep away from direct sun and rain.

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EN 4. Instructions for using the product:

• Carry out prior selection of the Transfer required for the impression technique, molding type and component to be used.

• Implacil Mold Transfers come in a blister-pack sealed with Tyvek. The Blister is then placed inside a cardboard carton.

• Check that the packaging is in perfect condition. Do not use the product if the packaging is damaged.

• Open the outer packaging. Remove the blister from the box and check that it is perfectly sealed.

• Remove the Transfer from the blister.

• The product is not pre-sterilized and sterilization procedures must be carried out by the dental surgeon in accordance with the current norms (NBR 14332), or any current legislation or norm that may come to complement or substitute the same.

• Screw the Mold Transfer onto the component you wish to transfer to the model.

• Use the impression technique to obtain a mold with the exact position of the Mold Transfer.

• Remove the Mold Transfer from the patient's mouth and fit it into its place in the mold obtained.

• Fit the analog of the component to be transferred into the mold, screwing it into the Mold Transfer.

• Pour the plaster to obtain the working model containing the clone of the component in its original position in the patient's mouth.

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EN 5. Warnings and/or precautions to be taken: The use of Implacil Mold Transfers should only be carried out by qualified and specialized professionals. The dental surgeon is question must draw up a prosthetic procedure plan. Implacil Mold Transfers are designed for single usage and re-processing of the materials is strictly prohibited. It must not be used, under any circumstances, if the packaging has been violated. The product is not pre-sterilized and sterilization procedures must be carried out by the dental surgeon prior to any application on patients. Given the small dimensions of the Mold Transfer, the dental surgeon must take the utmost care to avoid its being swallowed or breathed in by the patient. The dental surgeon must pay attention to the amount of force used when applying the product in order to avoid damage to the same. It is up to the dental surgeon to assess the quality of the initial stability so as to determine whether the implants are ready for load bearing.

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EN 6. Undesired secondary or side effects and contraindications: If it is not used according to the correct technique the end result of the application may not be successful, leading to loss or fracturing of the product.

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EN 7. Care to be taken in disposal of product: To avoid environmental contamination or inappropriate use of the Mold Transfers, after utilization they must be disposed of in a suitable location, in other words, in a place for potentially contaminated materials.

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EN 9. Term of Guarantee: IMPLACIL DE BORTOLI MATERIAL ODONTOLÓGICO LTDA. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. Provided it is suitably transported, stored and packaged, in accordance with these Instructions for Use, the product will have an indefinite shelf-life.

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