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EN Instruções for product use: The Surgical Instruments should be used to assist in dental surgery. They should be sanitized and sterilized beforehand according to the following techniques:

A) Cleaning: The cleaning of a surgical instrument that has been in contact with tissues should be executed in such a way that all the parts of the item are exposed to the cleaning process. This process should involve an initial rinse of the item with cold water, washing with neutral to slightly alkaline detergent, with low foam formation, a rinse with water (from the faucet or conditioned) followed by a final rinse with deionized or reverse osmosis purified water. The initial cold rinse is performed to remove any gross contamination present in the item. Cold water is used, since temperatures above 140 F will coagulate proteins, making it difficult to remove them from the contaminated items. After the cold water rinse, the items should be cleaned with a neutral to slightly alkaline detergent with low foam formation, and pH between 7.0 and 10.0. If the contamination contains heavy organic residues (protein or fat) or dry dirt, an enzymatic detergent may be used. The detergent used should also be easy to rinse and not leave residual compounds. A detergent with low foam formation reduces the formation of aerosol droplets, which may transport microorganisms. Detergent residues can cause stains and interfere in the action of chemical disinfectants. The washed items should be rinsed to remove the detergent, preferably with a final rinse in deionized or reverse osmosis purified water. The cleaning process should be executed in such a way that all the parts of the surgical instrument are exposed. Contamination remaining in an item after cleaning reduces the efficacy of any subsequent decontamination or sterilization process.

Clean the instruments as soon as possible after use. Do not allow blood and residues to dry on the instruments. If the cleaning has to be postponed, place groups of instruments in a container covered with enzymatic solution or appropriate detergent in order to delay drying.

B) Sterilization: Sterilization is a procedure designed for total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form). After the drying process, the instruments are sent for sterilization in an autoclave, using temperature and exposure time according to the following method: sterilization temperature: 132°C, sterilization cycle: 20 minutes and drying cycle: 20 - 60 minutes. The surgical instruments should be arranged inside the container (with perforated lid and bottom), or tray or in the box; provided they are accommodated in such a way as to allow their preparation for the sterilization process. DO NOT OPEN the autoclave to avoid rapid condensation; in other words, let the steam out first, so that the Drying Cycle terminates on its own. Clean the autoclave rigorously and periodically, removing the dirt and excess rust formed (iron oxide).

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EN Warnings and/or precautions to be adopted: Particular care should be taken in the cleaning, sterilization, accommodation (stockpiling), transportation and handling of the product to avoid mechanical shocks that change its anatomy and affect its functionality. Before the start of the surgery verify the product operation. The product should not be used if it exhibits anatomical alterations. When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are not sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box pursuant to item 4b. Instrument boxes removed from their packaging do not remain sterile. The Surgical Instruments should be replaced as soon as the professional perceives their diminished efficacy. For exclusive use by dental professionals.

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• Due to necessary mechanical functionalities, most instruments are made from non-implantable materials. In the eventuality of an instrument breaking, no fragment should remain in the patient, as it may cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metal components, possibly requiring subsequent intervention.

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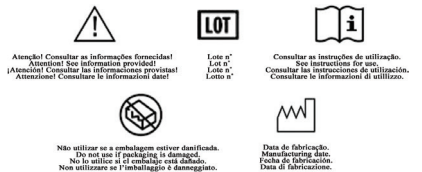
EN Precautions in case of disposal: To avoid environmental contamination or improper use of the Surgical Instruments, these must be disposed of in an appropriate place, that is, in a specific place for potentially contaminated materials.

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Nota: a garantia não cobre desgaste de uso do produto. EN Product warranty: IMPLACIL DE BORTOLI MATERIAL ODONTOLOGICO LTDA, assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

N.B.: the warranty does not cover wear and tear from use of the product.

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